

EYEGIENE®

Sterile, eye cleansing wipe

Features:

- Soft, disposable, sterile wipes for daily cleansing, hydration and soothing of the internal and external eyelids and the periocular area.
- Premoistened with an isotonic, water based solution containing ingredients of proven efficacy.
- Non allergenic solution, dermatologically tested.
- Doesn't cause opthalmic irritation or sensitization, even when used regularly.

INGREDIENTS	PROPERTIES
<i>Hyaluronic acid</i> (<i>Sodium Hyaluronate</i>)	Moisturizing, hydrating and relaxing properties
<i>Linoleamidopropyl</i> <i>Pg-Dimonium Chloride</i> <i>Phosphate (phospholipid)</i>	Favours a deep but non aggressive cleansing of the eye
<i>Chamomilla recutita</i>	Soothing, decongestant, anti-inflammatory, refreshing properties
<i>Helichrysum italicum</i>	Anti-inflammatory, antioxidant (photo-protective action against UV rays) and decongestant properties

- Eyegiene® does not burn or cause pain, does not irritate the eye, does not interfere with other ophthalmic products.

Area of use:

- Daily hygiene of the eyes and the peri-ocular area.
- In case of thyroid disorders, sty, chalazion, blepharitis.
- Treatment of allergic conjunctivitis infections of various kinds even in new born babies.
- Daily hygiene of patients suffering from ophthalmological implications associated with diabetes and lack of A, B, PP vitamins.
- Hypersensitivity due to the environmental agents, such as dust, smog, pollen, long exposure to screen.
- Reduction of the sensation of fatigue, foreign bodies in the eyes and dryness due to the lax of tears.
- Before/after ophthalmic surgery.
- Suggested for contact lens wearers.
- Recommended for adults and children

Classification:

- Medical Device Class IIa
- Sterile
- Size of the wipe 14 cm x 15 cm



CE
0477

EYE GIENE®

Sterile, eye cleansing wipe

Instructions:

- *Self-medication:* Open the sachet using the easy-opening edge. Remove the wipe and gently clean inside the eyelids and the area around the eyes. Use a separate wipe for each eye. Change the wipe after cleansing an infected area in order to avoid the contamination of healthy areas. Do not rinse or dry the eye after using the wipe.
- *Professional use:* Open the sachet using the easy-opening edge, designed for use onto a sterile field. Remove the wipe and gently clean inside the eyelids and the area around the eyes. Use a separate wipe for each eye. Change the wipe after cleansing an infected area in order to avoid the contamination of healthy areas. Do not rinse or dry the eye after using the wipe.

Certification:

- Welcare is certificated:
UNI CEI EN ISO 13485
UNI EN ISO 9001
UNI EN ISO 14001

Production flow:

- Manufactured in ISO 7 clean room.

Code:	Description:	Sales Units:
<i>95250S</i>	<i>Eyegiene® 1 pouch: 1 disposable sterile wipe 1 dispenser: 40 pouches</i>	<i>1 shipping box: 12 dispensers</i>

Further information:

Website: www.welcareindustries.com

Customer service: com@welcaremedical.com

Welcare Industries S.p.A.
Orvieto, Italia.
www.welcareindustries.com