

SINAQUA®

Shampoo Cap

*Medical device
for the prevention
and mechanical
treatment of mycosis
of the scalp*

Features:

Disposable cap, waterproof externally and coated internally with a soft, pre-moistened cloth. Provides a convenient tool for the hygiene of scalp and hair in non-self-sufficient and bed-ridden persons.

The solution, containing Aloe Vera leaf extract, Phospholipids, Panthenol leaves the hair soft and manageable. Sinaqua® Shampoo Cap does not require the use of water.

No rinsing.

Hair will be soft and manageable. The product does not contain parabens, urea, PEG, glycerin, alcohol, latex.

Classification:

- Medical Device Class I



Active ingredients:

- *Aloe barbadensis leaf extract*
- *Sodium Coco pg-dimonium chloride phosphate (Phospholipid CDM)*
- *Panthenol*

Technical features:

- Waterproof externally.
- Elastic perimeter.
- Tissue in viscose to provide:
 - High absorbency of skin impurities.
 - High resistance to stretch and torsion to avoid tearing of the internal liner.
 - Maximum softness to avoid skin irritation.
- High concentration of cleansing solution in each cap (150 ml).

Certification:

- Welcare is certificated:
UNI CEI EN ISO 13485
UNI EN ISO 9001
UNI EN ISO 14001



SINAQUA[®]

Shampoo Cap

*Medical device
for the prevention
and mechanical
treatment of mycosis
of the scalp*

Instructions:

- Sinaqua[®] Shampoo Cap, closed in its bag, can be heated to the desired temperature in a microwave oven for no more than 15 seconds or the bag can be put in hot water.
- Place the cap on the head.
- Massage for a few minutes.
- Remove the cap and do not rinse.
- Dry the hair with the traditional method used.
- Make sure not to over-heat Sinaqua[®] Shampoo Cap.

Production flow:

- Manufactured in ISO 7 clean room

<i>Code:</i>	<i>Description:</i>	<i>Sales Units:</i>
<i>95200</i>	<i>Sinaqua[®] Shampoo Cap 1 bag: 1 cap</i>	<i>1 shipping box: 30 bags</i>

Further information:

Website: www.welcareindustries.com

Customer service: com@welcaremedical.com